



.OPERATING TABLE

UNIVERSAL OPERATING TABLE SU-02 is designed for carrying out treatments and operations in the range of general surgery and also using the appropriate equipment for orthopaedy, proctology, gynaecology, urology, ophthalmology, laryngology and neurosurgery.

The table tops of the SU-02 operating table are made of stainless and acidproof steel. They have removable, seamless, anti-static polyurethane mattresses. Back rest, head rest and leg segment adjustment are supported by gas springs with interlocking. Gear racks regulate the two-section leg rest. Convex inclination of the divided back rest segment is actuated manually by a worm gear operated by a crank. This model is existing in 3 models, 6, 5 & 4 sections and in 2 presentations: with big castors or with steering from patient's head side



Version of operating table SU-02 with big castors

All metal elements which are exposed to external factors are made of stainless, acid-proof steel. Adjustment of the table top's height is performed thanks to hydraulic actuator and foot operated hydraulic pump. The table base is movable and it is equipped with central wheels brake set via foot pedal. The adjustment of the back rest, leg rest, head rest and Trendelenburg and reverse Trendelenburg positions are facilitated thanks to gas springs with interlocking set via levers, which are placed in places convenient for operator. Lateral tilts are accomplished via crank mounted underneath the seat segment. Table tops are equipped with mattresses, which are resistant to disinfectants. Table top's segments are translucent to X-rays and enable taking pictures and monitoring patient with C-arm. C-arm monitoring is restricted with the table's column width. Chest, gall bladder operations Down and aside adjustable leg rests enable anatomic arrangement of patient for laparoscopy treatment Head, neck operations Lateral position (kidney operations, thorakosurgery)

- Adjustment of leg rest's inclination
- Adjustment of leg rest's opening and blocking
- Lateral tilts regulation
- Hydraulic pump lever
- Brake lever
- Anti-static mattress made of polyurethane foam
- Kidney elevator
- Adjustment crank
- Head rest SG-10.0 adjusted thanks to gas springs
- Adjustment of Trendelenburg and reverse Trendelenburg position
- Adjustment of head rest
- Adjustment of back rest
- Head rest blocking knob
- Potential equalization socket



Version of operating table SU-02 with steering from patient's head side

The product complies with the European Directive MDD 93/42/EEC relating to the medical devices safety



.OPERATING TABLE

TECHNICAL DATA:

Total table length with 4 or 5 sections table top 2025 mm

Total table length with 6 sections table top 2160 mm

Total table top's width 550 mm

Table top's height 750 to 1000 mm

Trendelenburg position 30 °

Reverse Trendelenburg position 25 °

Back rest inclination -40° to 70 °

Head rest inclination -20° to 50 °

Lateral tilts 25 °

Table weight 170 kg

RECOMMENDED EQUIPMENT:

1. Drip bottle holder WK-01.5
2. Angled arm support WS-07.5
3. Anaesthetic frame WS-01.5
4. Shank rests WS-04.5
5. Hand rests WS-02.5
6. Knee rests WS-05.5
7. Holder for anaesthesia tubes WS-14.5
8. Shoulder rests left and right WS-06.5
9. Thigh rests WS-03.5
10. Arm support PR-01.5
11. Big side rests WS-08.5
12. X-ray plate tray WS-11.5
13. Tray for instruments WS-15.5
14. Gynaecological attachment WS-28.5
15. Trolley for accessories WS-13.5

ADDITIONAL EQUIPMENT:

- Orthopaedic attachment SO-11
- X-ray plate grip for back rest WS-19.6
- Laryngological-ophthalmic head rest WS-21.5
- Hand support for a doctor WS-22.5
- Urological attachment WS-29.5
- Proctological attachment WS-30.5
- Wristlet WS-34.5
- Attachment for arthroscopy treatment WS-39.5
- Attachment for meniscus treatment WS-40.5
- Abdominal strap WS-41.0
- Patient restraint strap WS-42.0
- Hand traction device WS-47.5
- Hand operating table WS-48.5
- Neurosurgical attachment Doro



ADDITIONAL EQUIPMENT: